

## STAPHYLO LA "SEIKEN" POSITIVE CONTROL

This control contains inactivated, lyophilized *Staphylococcus aureus* cells.

### INTENDED USE

To confirm positive test results when using STAPHYLO LA "SEIKEN."

### METHOD OF USE

1. Add 0.5 mL saline to the vial and suspend well.
2. Place one drop of this suspension on the agglutination plate.
3. Add one drop of the sensitized latex reagent to the suspension.
4. Mix well with the stirrer and shake the plate manually back and forth for 20 seconds while observing the progress of the agglutination.
5. Interpret immediately by visual inspection under sufficient light.
6. Refer to STAPHYLO LA "SEIKEN" instruction sheet for the details.

### PRECAUTIONS

1. Store reconstituted suspension at -20 °C. Avoid repeated freezing and thawing.
2. Although the *S. aureus* cells contained in this reagent have been inactivated, they should be treated as living cells and handled with appropriate precautions.

### ※ STORAGE AND SHELF LIFE

Storage: 2 - 10°C protected from light.

Shelf life: Up to the expiry date on the label.

### PACKAGE

STAPHYLO LA "SEIKEN" POSITIVE CONTROL (lyophilized)  
for 0.5 mL x 1 vial